

ACTION TRACKCHAIR OWNERS MANUAL



Action Manufacturing
1103 Canoga Park Drive
Marshall, MN 56258
507-532-9649

www.actiontrackchair.com
tim@actiontrackchair.com

1. INTRODUCTION
2. SAFETY
3. OPERATING THE ACTION TRACKCHAIR
4. ASSEMBLY
5. COMFORT ADJUSTMENTS
6. BATTERIES AND CHARGING
7. CARE AND MAINTENANCE
8. WARRANTY
9. SPECIFICATIONS
10. DEMO FORM

INTRODUCTION

Welcome to Action Trackchair. We at Action Trackchair want to make your experience the best it can be. Enclosed in this owner's manual you'll find information to use and maintain your Action Trackchair. With any questions please contact us at:

Action Manufacturing

1103 Canoga Park Dr

PO Box 587

Marshall, MN 56258

507-532-9649

tim@actiontrackchair.com

SAFETY GUIDELINES

- Only one person should be on the trackchair at any time.
- Seat belt is recommended.
- Trackchair will climb inclines enough to tip over in any direction.
- Make sure controls are in the off position before sitting in trackchair and before getting out of seat.
- Always have a backup plan, "What if?"
- Do not ride the trackchair during loading or unloading from vehicle or carrier.
- Do not attempt to climb stairways.
-

OPERATING YOUR ACTION TRACKCHAIR

- When you are ready to drive the trackchair, make sure controls are in the off position before sitting in trackchair.
- If your trackchair has a locking control, it can be unlocked in this way. Turn control on, hold joystick forward until you hear a beep or three seconds, then joystick back until you hear a beep or three seconds. It is now unlocked and ready for operation.
- When locking the control, it is done in this way. After the control has been turned off, hold the on/off button until the control has cycled both on and then off. Control is now set in the locked mode.
- The Action Trackchair control has five speeds, one-five and can be changed with the up and down arrows.
- Battery indicator is on the main screen on controls. Battery charge will last up to six hours, depending on battery condition and type of use the trackchair is subject to. Action Trackchair has a built-in battery charger the plugs into 110 voltage.
- Optional lighting is available and is controlled on the control panel.
- Manual tilting of the chair is possible by pushing the “M” button on controls, and then moving joy stick forward or backward to tilt chair. Cancel by pushing “M” again or moving joystick to the left.
- If for some reason it is necessary to pull the trackchair. Disengage the brakes on the motors with the levers on back of motors. Push lever to the outsides on both motors. Do not pull trackchair more than 5 MPH

ASSEMBLY

When you receive the trackchair it will be fully assembled. Just make sure battery is charged and it will be ready to go.

COMFORT ADJUSTMENTS

There are few adjustments that are necessary. The leg rest can be moved up or down to fit the riders needs. The chair itself can be leveled to the desired comfort of the rider. Electronic controls can be adjusted at a servicing dealer as far as speed, acceleration, deceleration, braking, etc.

BATTERIES AND CHARGING

- Battery charge will last up to six hours, depending on battery condition and type of use the trackchair is subject to. Trackchair has a built-in battery charger the plugs into 110 voltage.

REPAIRS AND MAINTENANCE

- All bearings are sealed and need no additional greasing.
- Track can be adjusted by loosening both bolts on the front idlers. Track tensioners can be tightened with a 9/16 wrenches by holding the lock nut and turning track tensioner bolts. It is not necessary to over tighten track.

WARRANTY

- Batteries One year
- Controls One year
- Tracks One year
- Motors One year
- Drive components One year
- Seat frame welding Five years
- Track frame welding Five years
- All other parts One year

SPECIFICATIONS

Height	38 inches
Width	37, 39, 41, or 43 inches
Length	48 inches w/rear idlers
Weight	350 pounds
Seat height	24"
Tilt angle for chair	20 degrees each way
Track Size	6.5" X 90"
Batteries	Two 12 volt Wheelchair Batteries
Controls	Action Trackchair controls
Motors	24 volt DC 18:1 ratio Standard 24 volt DC 24:1 ratio high thrust
Speed	4-6 MPH
Turning Radius	ZERO
Width between armrests	18, 20, 22 or 24 inches
Ground Clearance	3.5 inches
Battery Charger	8 Amp
Range	Variable up to 10 Miles
Foot rest	Adjustable Standard or Flip up
Accessory holders	two on each side and two on back of chair
Lap belt	Standard

DEMO FORM

Date _____

Location _____

Safety instructions:

- Only one person on trackchair at a time.
- Know the limits of the trackchair.
- The trackchair will climb an incline high enough to tip over.
- Failure to know limits can cause personal injury or equipment damage.
- Always have an able-bodied person close enough to help out in need.
- Do not ride on trackchair as loading on trailer or vehicle.

Ownersmanualdemoform.doc